Hybrid Foam Block

INSTRUCTION BOOKLET

Walkease Hybrid Foam Blocks are the fast, cost-effective solution for managing moderate lameness in cattle.

Attaching a Walkease Hybrid Foam Block to the healthy claw provides elevation for the injured claw, promoting healing by minimising weight bearing and pressure. With no removal required, Walkease Hybrid Foam Blocks speed up recovery and mobility keeping your herd productive.

ENGLISH	1	FRANÇAIS	6	NEDERLANDS	11	SUOMI	16
中文	2	ITALIANO	7	NORSK	12	SVENSKA	17
DANSK	3	日本語	8	POLSKI	13	TÜRKÇE	18
DEUTSCH	4	LIETUVIŲ	9	PORTUGUÊS	14		
ESPAÑOL	5	MAGYAR	10	РУССКИЙ	15		

Always use with Walkease Rapid Glue for best results.

Shoof Lameness Scale

A cow's lameness is assessed on a scale of 0 – 3, with 0 representing a normal walk and 3 indicating a severely lame cow.

Assessment is based on indicators such as walking speed, rhythm, stride length, weight bearing, back alignment, and head movement.

Score	Action	Walkease Foam Blocks	Walkease Hybrid Blocks	Walkease Wooden Blocks	Hoofers Cow Clogs
0 No Lameness	Walks evenly, no action required				
1 Mild Lameness	Minor action required, monitor	✓			
2 Moderate Lameness	Action required, examine within 24 hours		✓	✓	
3 Severe Lameness	Urgent action required, treat immediately				✓

Need help? We're here for you.

New Zealand: Shoof International Ltd. 224 Laurent Road North, Cambridge 3493, New Zealand. Ph 0800 800 801

Australia: Shoof International Pty Ltd. 1 International Square, Tullamarine VIC 3043, Australia. Ph 1800 121 801

Chile: Shoof International Chile Ltda. Loteo Las 7 Colinas, Fundo el Castillo, Lote N° 15, Barrio Industrial Osorno, Chile. Ph 800 600 000

www.shoofint.com/walkease

Shoof® and Walkease® are registered trademarks of Shoof International Ltd.

Scan to view Video

Walkease®: Trusted hoof care solutions

Proudly designed and developed in New Zealand by Shoof.

Walkease Hybrid Foam Blocks are made with a specially-formulated ethylene-vinyl acetate (EVA) foam outer and wooden core. EVA offers a lightweight, flexible construction with superior shock absorption and long-lasting durability. The wooden core provides superior elevation and lasts up to 28 days. They are attached to the hoof by using Walkease Rapid Glue and can be used on either the left or right claw. Walkease Hybrid Foam Blocks are low-cost and easy to apply. Special skill and experience are not required. The directions in this instruction booklet must be adhered to. Any deviation from these instructions will likely result in failure of the adhesive.

Walkease Hybrid Foam Blocks are intended for use to repair hoof injuries resulting in moderate lameness. They are ideal for the recovery of such ailments as thin soles, and other conditions needing about 2 - 4 weeks of elevation.

Walkease Rapid Glue is different to most other hoof-block adhesives. The natural moisture in the hoof and in the atmosphere is required to activate the glue. **The hoof must not be artificially dried (hot air or alcohol).** In very cold weather, when the air is very dry, more time might be required to activate the glue. (Breathing closely on the glued surface a few times before application to the hoof will help.)

To maintain shelf life the glue should be stored in a refrigerator. Remove from the fridge prior to use. The glue is best used at about room or body temperature.

Fitting procedure:

Treatment of the injured claw must be completed, then the healthy claw is prepared for fitting the block. The Walkease rasp should be used to finish the hoof surface regardless of what other tools may have been used to prepare the hoof. This rasp provides the correct clean and flat surface for the Walkease Hybrid Foam Block to adhere to. The rasp must be kept clean.

Handle the Walkease Hybrid Foam Block carefully ensuring the gluing face is kept clean. Test the block for size and shape on the hoof. Choose the correct size that will support all of the hoof wall, but without creating any large overlap.

Always wear gloves when handling Walkease Rapid Glue. Apply glue using the full contents of the 2 mL single-use tube to secure the block. After about 10 seconds of steady pressure you will feel the block 'grip'. (Possibly longer in cold weather). Promptly after 'grip', lower the hoof so the animal can put full weight on the block to complete the bond. The Walkease Hybrid Foam Block will not attach properly if weight is not applied before the glue has completely cured.

The glue sets very quickly under the animal's weight. The excess glue is squeezed out and the block conforms to the hoof surface. This improves the surface area contact giving a stronger bond. In cold weather the bond time may increase

a little, so give the animal a bit longer standing on the block before releasing it from the crush.

Walkease Hybrid Foam Blocks do not generally need to be removed. They erode naturally after 2 - 4 weeks.

Extra blocks and glue can be ordered through your ShooF dealer.



Fig 1. Treat the injured claw as required.

Fig 2. Use the rasp to prepare the adjacent claw for gluing.

Fig 3. Apply full tube of glue to the Walkease Hybrid Foam Block.

Fig 4. Press the block firmly onto the healthy claw, overlapping the hoof wall.

Fig 5. Promptly drop the leg for the cattle to put weight on the block.

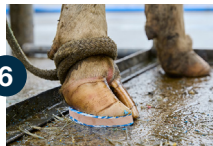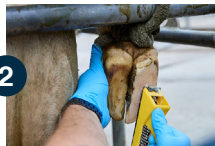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