Wooden Block

INSTRUCTION BOOKLET

Walkease Wooden Blocks provide a long-lasting solution for managing moderate lameness in cattle.

Attaching a Walkease Wooden Block to the healthy claw provides elevation for the injured claw, promoting healing by minimising weight-bearing and pressure for up to 35 days. Walkease Wooden Blocks speed up recovery and mobility, keeping your herd mobile and productive. A wooden block may shed naturally or require removal.

Available in 18 languages

ENGLISH
中文
DANSK
DEUTSCH
ESPAÑOL

FRANÇAIS
ITALIANO
日本語
LIETUVIŲ
MAGYAR

NEDERLANDS
NORSK
POLSKI
PORTUGUÊS
РУССКИЙ

SUOMI
SVENSKA
TÜRKÇE

Always use with Walkease 2-Part Glue and Walkease Glue Dispenser Gun for best results.

Shoof Lameness Scale

A cow's lameness is assessed on a scale of 0 – 3, with 0 representing a normal walk and 3 indicating a severely lame cow.

Assessment is based on indicators such as walking speed, rhythm, stride length, weight bearing, back alignment, and head movement.

Score	Action	Walkease Foam Blocks	Walkease Hybrid Blocks	Walkease Wooden Blocks	Hoofers Cow Clogs
0 No Lameness	Walks evenly, no action required				
1 Mild Lameness	Minor action required, monitor	✓			
2 Moderate Lameness	Action required, examine within 24 hours		✓	✓	
3 Severe Lameness	Urgent action required, treat immediately				✓

Need help? We're here for you.

New Zealand: Shoof International Ltd. 224 Laurent Road North, Cambridge 3493, New Zealand. Ph 0800 800 801

Australia: Shoof International Pty Ltd. 1 International Square, Tullamarine VIC 3043, Australia. Ph 1800 121 801

Chile: Shoof International Chile Ltda. Loteo Las 7 Colinas, Fundo el Castillo, Lote N° 15, Barrio Industrial Osorno, Chile. Ph 800 600 000

www.shoofint.com/walkease

Shoof® and Walkease® are registered trademarks of Shoof International Ltd.

Walkease®: Trusted hoof care solutions

Proudly designed and developed in New Zealand by Shoof.

Walkease Wooden Blocks are part of the Walkease hoof care system offering reliable, longer-term claw elevation for moderate lameness, when durability and extended support are required.

Walkease Wooden Blocks feature a grid pattern designed for use with Walkease 2-Part Glue, our patented polyurethane adhesive. This grid design helps to reduce shear forces when the cow twists or turns, allowing the adhesive to spread evenly and form a strong, interlocked bond with the claw.

Fitting Walkease Wooden Blocks

Correct preparation and application are essential to ensure maximum bond strength and block retention.

Treat the affected claw

Trim and treat the lame claw. Proper hoof preparation is essential.

Prepare the healthy claw

Thoroughly clean the healthy (weight-bearing) claw. The healthy claw surface should be clean and dry to ensure effective bonding. Inadequate drying may lead to poor adhesion.

Walkease 2-Part Glue application

Insert the Walkease 2-Part Glue cartridge into the Walkease Glue Dispenser Gun and attach a new mixing tip. The mixing tip ensures that the two parts of the polyurethane adhesive mixture are dispensed at the correct ratio.

Apply the block

Using the Walkease Glue Dispenser Gun, begin applying glue to the wooden block. Ensure both components flow evenly.

IMPORTANT: Apply and position the block immediately after mixing, as the adhesive begins curing approximately 12 seconds after mixing.

Apply approximately 20 mL of the mixed adhesive, in a continuous zig-zag pattern across the block (Fig. 3), to ensure even coverage when pressed onto the hoof, particularly around the edges, see (Fig. 4) below.

Press the block firmly onto the healthy claw, ensuring it is correctly aligned and the glue spreads to cover the entire block surface. Maintain firm pressure for approximately 30 seconds.

Full bond strength develops as the adhesive cures and becomes fully load-bearing in approximately 90 seconds, depending on environmental temperatures.

Once set, gently lower the foot allowing the animal to place weight on the block.

Walkease Wooden Blocks are designed to wear down naturally and typically detach as the hoof grows, but may require removal if they do not naturally wear off. Blocks should not be left on a claw for longer than 35 days.

We recommend using Walkease 2-Part Glue and the Walkease Glue Dispenser Gun with wooden blocks for best performance.

Note: In cold conditions, bond time may vary. Store cartridges at room temperature before use to improve flow and curing.

- Fig 1. Treat the injured claw as required.
- Fig 2. Apply approximately 20 mL of the mixed adhesive, in a continuous zig-zag pattern across the block to ensure even coverage.
- Fig 3. Ensure full coverage and that the block is properly aligned with the claw axis before placement.
- Fig 4. Press the block firmly onto the healthy claw. Maintain firm pressure for approximately 30 seconds.
- Fig 5. The Walkease Wooden Block becomes fully load-bearing in approximately 90 seconds. Once set, gently lower the leg allowing the animal to place weight on the block.

Additional Wooden Blocks, Mixing Tips and 2-Part Glue can be ordered through your Shoof dealer.

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